




Instructions

SMT baby[®]


SMT môm[®]



Position each of the nose guide's 6 noses on the child's nose.
Choose the nose best suited to the morphology of the child's face.
Each nose is held in place on the guide via two attachment points.
Detach the nose chosen from the guide.

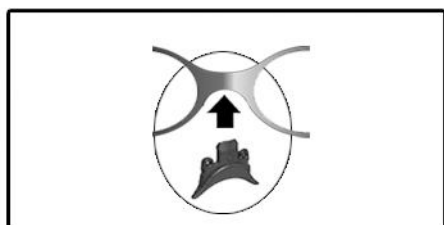


Carefully insert the nose into the frame, by hand or using a pair of suitable pliers.

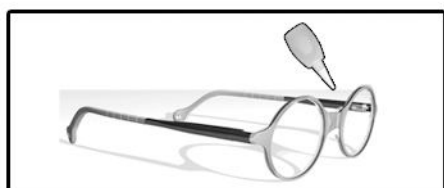

If necessary, place a spot of glue on the top surface of the nasal notch, once the nose is in position.



To remove the nose from the mounting, use the pliers positioned in the middle of the nose guide.
If you used a spot of glue, heat before using the pliers.

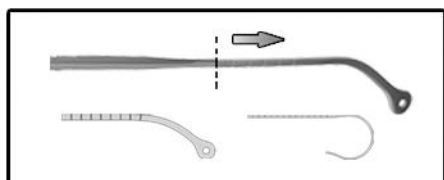

To shorten the arms: remove the sleeve, cut the sleeve and the metal part to the desired length.
It is essential to put a spot of glue between the sleeve and the plastic part of the arm.
You can also heat the arm so that the sleeve adheres to the stem in an optimum manner.



For SMT Môm frames :
If needed, loosen the cover by introducing a flat screwdriver between the cover and the bottom of the frame while turning the screwdriver.
The cover should be retightened with the careful use of a pair of flat nose pliers.